

Long Seat Sling



When you use a patient lift, you want the security of knowing that the slings you use are durable and well-made.

At Tollos, we're diligent about you and your patients' safety. So we're careful about ensuring the quality of the sling is up to your standards.

Slings are a Class I Medical Device. Tollos takes compliance with all applicable federal regulations very seriously.

We pride ourselves on our quality and your peace of mind.

Long Seat Sling

tollos

The Long Seat Sling is designed for general purpose use. Because this sling doesn't cross between the patient's knees, it's ideal for those with groin injuries or amputated limbs. It comes equipped with handles on the back for easy positioning of the patient.

The Long Seat Sling can be used for bed, chair, and floor lifting.

Long Seat Sling Features

- Seated sling designed for general lifting
- Ideal for amputees or groin injuries
- Comfortable, yet durable poly-blended fabrics or mesh
- Color-coded binding allows for quick sizing
- Available in washable reusables or in single-patient-use varieties

General Washing Instructions:

Note: Please refer to the enclosed instructions-for-use packaged with all slings.

All Tollos reusable slings should be laundered according to the instructions on the tag that has been affixed to each sling. The following washing instructions are to be followed for optimum performance to preserve and maintain the quality of Tollos reusable slings:

Spot Cleaning: Most disinfection solutions are suitable for spot cleaning the reusable slings & harnesses. Bleach products can be used in a dilution of 100ppm or 0.01%, however, greater concentrations may cause premature deterioration of the fabrics.

Reusable slings should be laundered between patients.

Washing Instructions:

Recommended wash cycle: BS 5651:1978 Hospital Laundry Wash (Normal)
Recommended detergent: ECE Reference Detergent + Sodium Perborate + TAED
Recommended Wash Cycle: 6 gallons (24 liters) of water at 165°F (74°C) for 15 minutes
Temp not to exceed 185°F (85°C).
Recommended Rinse Cycle: 4 cycles at 104°F (40°C) for 3 minutes per cycle
Recommended Dry Cycle: Drying cycle: BS EN ISO 6330:2012 Procedure E
Low Temp, not to exceed 120°F (50°C)

Bleach products may be used during the wash cycle in a dilution of 100ppm or 0.01%.

Do not launder single-patient-use slings.

Sling Guide

Reusable	Reusable Mesh	Single Patient Use	Size	Suitable for:	Binding Color	Safe Working Load
T015LPPF	T015LPPMF	T015LPP-PSS	X-Small	49 - 75 lbs	White	600 lbs/ 272 kg
T015LPPF	T015LPPMF	T015LP-PSS	Small	75 - 125 lbs	Red	600 lbs/ 272 kg
T015LFF	T015LMF	T015L-PSS	Medium	125 - 175 lbs	Yellow	600 lbs/ 272 kg
T015LXF	T015LXMF	T015LX-PSS	Large	174 - 250 lbs	Green	600 lbs/ 272 kg
T015LXXF	T015LXXMF	T015LXX-PSS	X-Large	250 - 400 lbs	Blue	1000 lbs/ 454 kg
T015LXXXF	T015LXXXMF	T015LXXX-PSS	XX-Large	400 - 600 lbs	Orange	1000 lbs/ 454 kg
T015LXXXXF	T015LXXXXMF	T015LXXXX-PSS	XXX-Large	600 - 1000 lbs	Gray	1000 lbs/ 454 kg



888.363.7224

www.tollos.com
LSSP 202006

